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Trouble 異常狀況	Cause 可能原因	Remedy 處理方式
The power of the equipment can not be turned on 設備的電源無法打開	Power cord and plug are not connected properly 電源線未正確連接插座	Connect them correctly 正確連接他們
	The built-in fuse is blown 內置保險絲燒毀	Replace the fuse with a new one 重新更換新保險絲
The lamp cannot be turned on , the cooling fan isn't start , the lamp switch red light is on 燈泡無法點亮，風扇未啟動，啟動開關紅燈亮起	The cooling fan or the power supply has failed 風扇或電源出現故障	Contact your sales representative 聯繫你的銷售代表
	The illuminator has failed 光源設備出現異常	Contact your sales representative 聯繫你的銷售代表
The lamp cannot be turned on , the cooling fan is operating , the lamp switch flashing red light 燈泡無法點亮，風扇正常啟動，啟動開關閃爍紅燈	The lamp has failed or life has expired 燈泡產生異常或壽命終止	Replace with a new lamp 更換新燈泡
	The lamp is not mounted properly 燈泡未正確安裝	Mount the lamp properly 正確安裝燈泡
The lamp can be turned on but is unstable 燈泡可點亮但不穩定	The lamp has failed or life has expired 燈泡產生異常或壽命終止	Replace with a new lamp 更換新燈泡
The lamp can be turned on but is dark 燈泡可點亮但未有光輸出	The lamp is not mounted properly 燈泡未正確安裝	Mount the lamp properly 正確安裝燈泡
	The light guide is not mounted properly 光導管未正確安裝	Mount the light guide properly 正確安裝光導管
	The dial value of light volume adjustment is close 光亮調節鈕被關閉	Rotate the light volume adjustment dial toward 100% by each level 按各階段光亮調整旋鈕至 100%
The lamp turns off during use 使用中燈泡關閉	The lamp has failed or life has expired 燈泡產生異常或壽命終止	Replace with a new lamp 更換新燈泡
	The built-in fuse is blown 內置保險絲燒毀	Replace the fuse with a new one 重新更換新保險絲

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